

◆◆ Safety Disposable Needles

◆ Product Use:


It can match with normal syringe in clinical injection of liquid medicine, which can effectively prevent the acupuncture accident.

Product Features:


1. It can match with all kinds of syringes with standard luer tip.
2. After injection, only need to push upward lightly or knock the safety protector, which can immediately achieve the purpose of protecting the needle tip.
3. Multiple protections, the safety protector will not be off accidentally.
4. Effective function and simple design can reduce the cost of safety injection.




Using step:

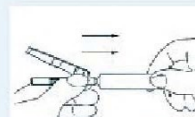

Assemble the safety needle on the luer tip of sterilized syringe.

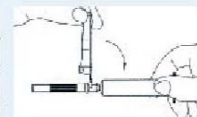

Open the safety protector, which will open automatically under the elastic force.

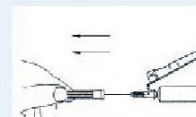

Take off the protector pull the plunger lightly to absorb the medicine liquid.



After injection, apply upward force on the safety protector, which will return automatically under the elastic force.



Apply force continually until the safety protector covers the needle.



Medical waste storage.

◆◆ Color codes and specifications for safety needle

SIZE		COLOR	LENGTH						ID		BEVEL ANGLE		REF
OD (mm)	GAUGE	COLOR & CODE	13 1/2"	16 5/8"	20 3/4"	25 1"	32 1 1/4"	38 1 1/2"	RW	TW	Long bevel	Short bevel	
0.45	26G	brown											SN-001
0.50	25G	orange											SN-002
0.55	24G	purple											SN-003
0.60	23G	blue											SN-004
0.70	22G	black											SN-005
0.80	21G	green											SN-006
0.90	20G	yellow											SN-007
1.10	19G	cream											SN-008
1.20	18G	pink											SN-009

Remark: normal size in above table, we can also make special size as clients' request.