

SW-800 Vision Screener

User's Manual

Step 1: Turn on/ Turn off the Unit



Please press the button for 2 Second,and then wait for 30 Second to enter the“Main Page”interface.

(For turn off: Please long press that button to turn off the unit)

Step 2: And then,Choose the “Screening Age range” to enter the measuring Mode:




PS:Please don t choose the “New”,it is just used for scanning the 2D code to lead out the patient Case.—Now it is just available to working with Data Cloud in China



Step 3: Measuring



The regular distance from the Patient to Vision Screener is **1M**, then it can capture the data automatically.

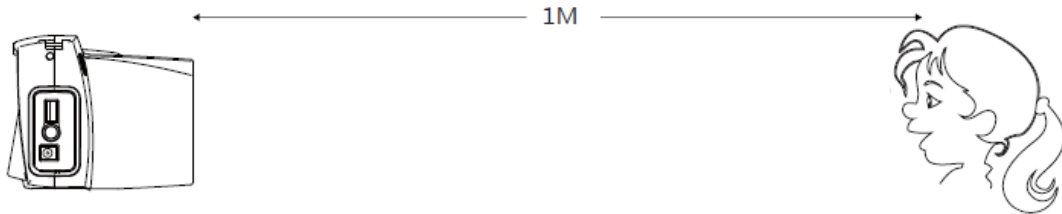


Please Keep on hold the Equipment on the process of measuring (as the pic showed as following):




and Please adjust(Fine tune)the distance according to the Prompts(Too Far or Near)

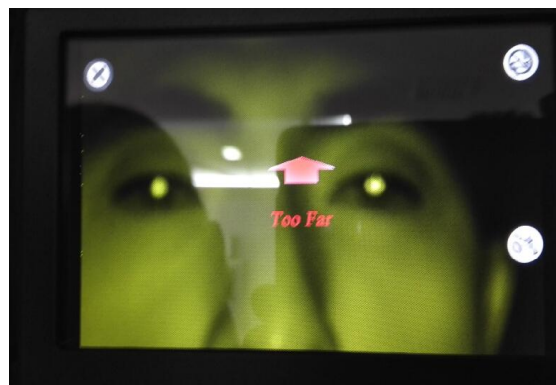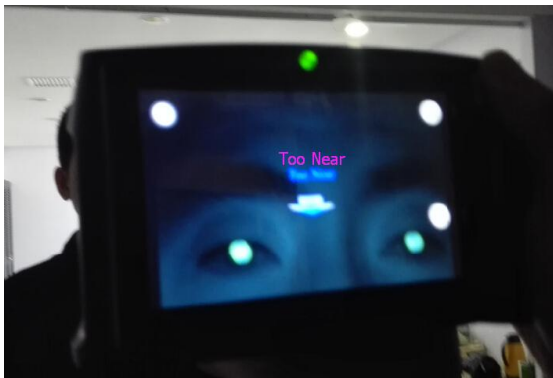

When the ring appears, it is proven that it is capturing data, so please don't move, until you get the final result.

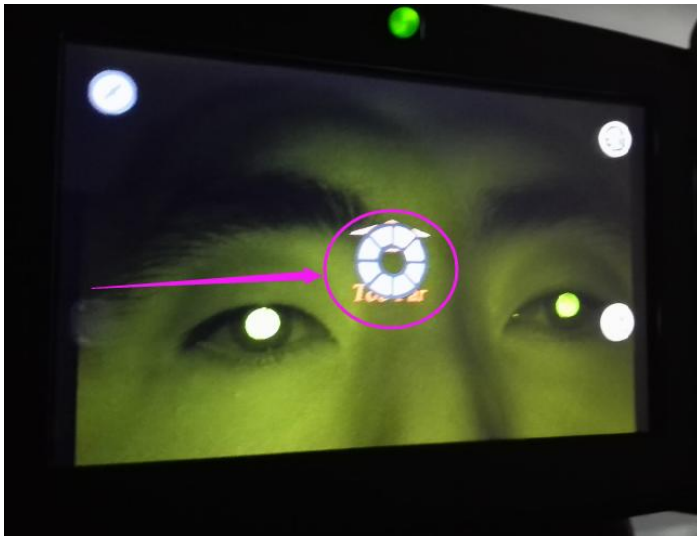

Normally it is just use 1-3 to capture the data.

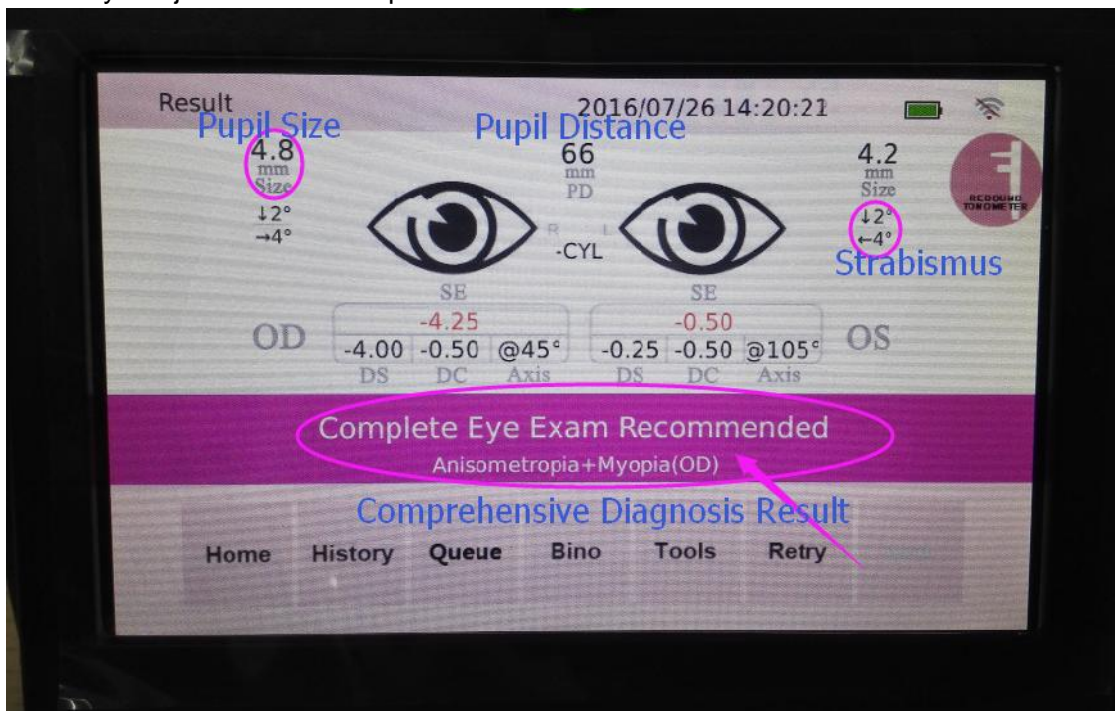

PS:

1. It is better to work in a dark room
2. Normally it is just available for the Pupil Size that more than 4MM

SE(Spherical Equivalent)	PD(Pupil Distance)
DC (Diopter of Cylindrical)	Pupil size
Axis (Axis angle)	


Step 4: Printing the result

How to setting the WIFI Printer:


1. Long press the button to turn on/off the WIFI Printer:




2. and then wait for 30 S ,until the WIFI Logo on the Screen of the Vision Screener change to Green..

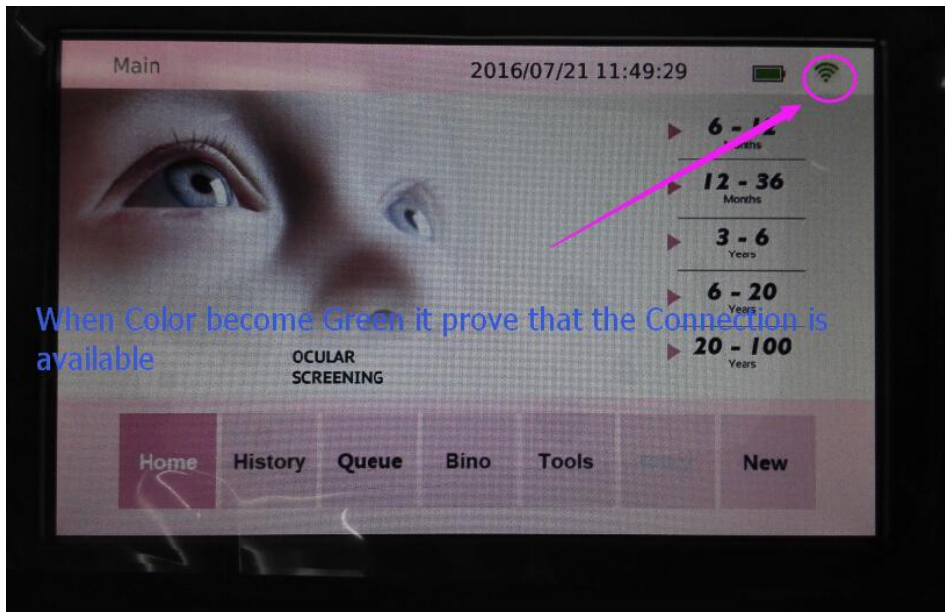


3. It is only can printing the data in this interface.

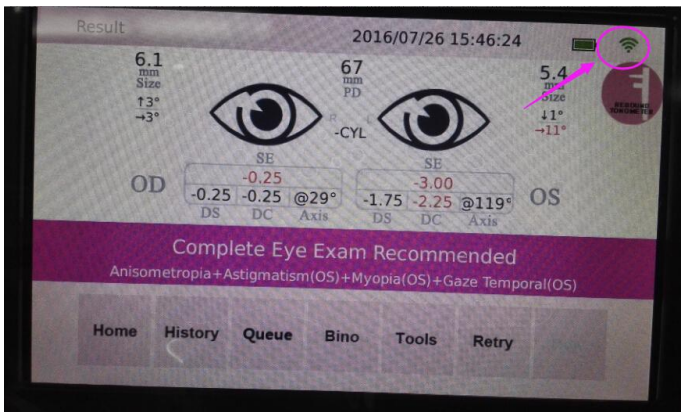

4. Press, Pink Block to print out the result.

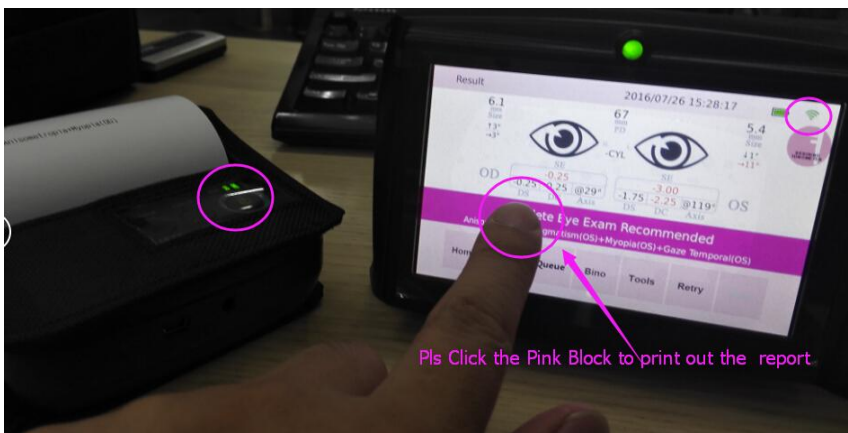


Report

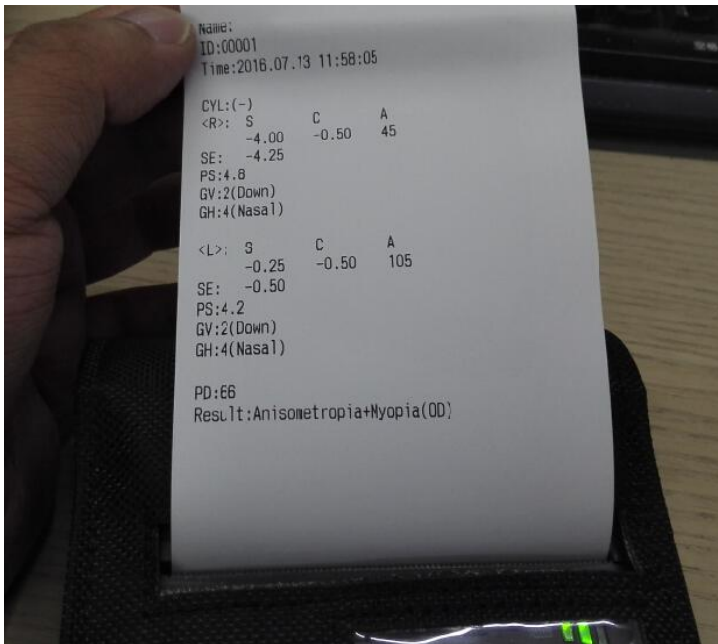



All measurement result can be saved on the Screener automatically.




For Edit the Patient Case:


Click the “**History**”Block to enter the following interface:

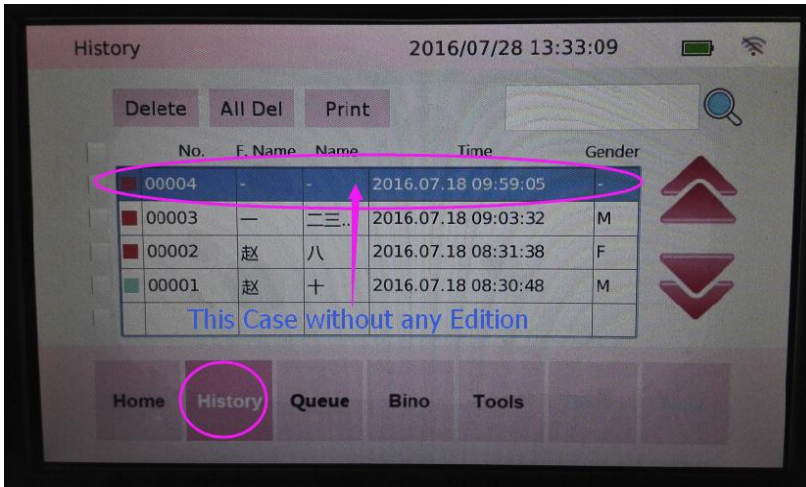


Edit the patient information:

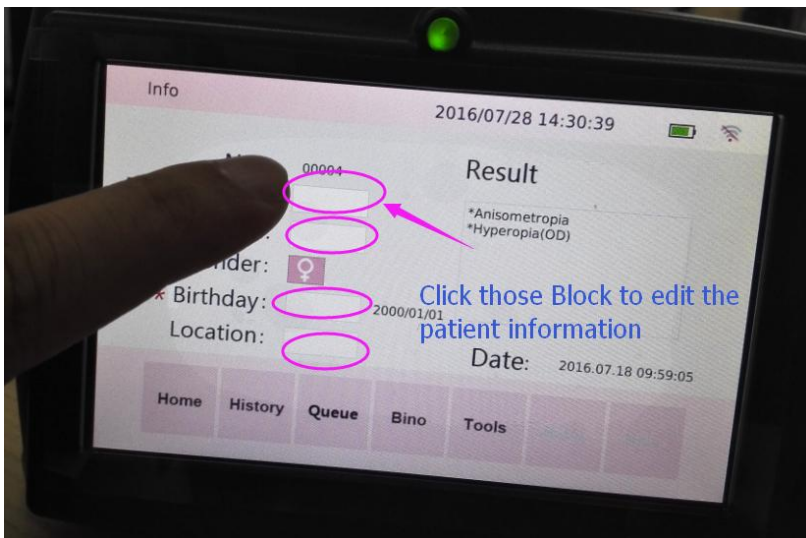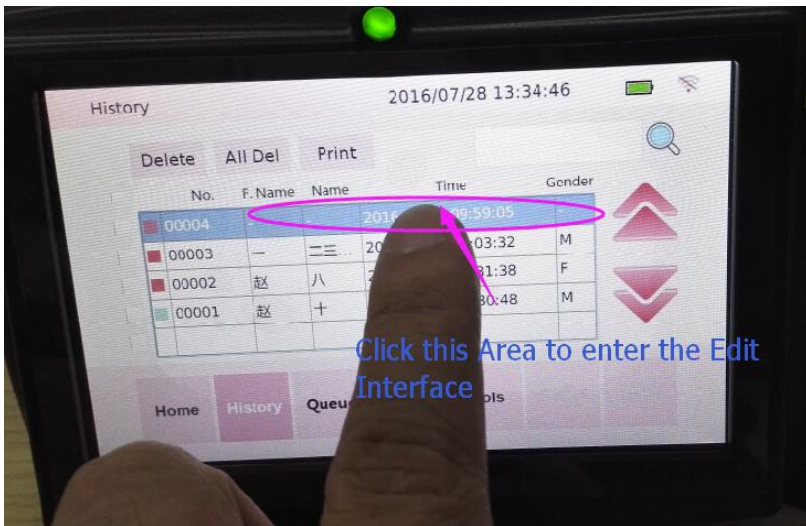

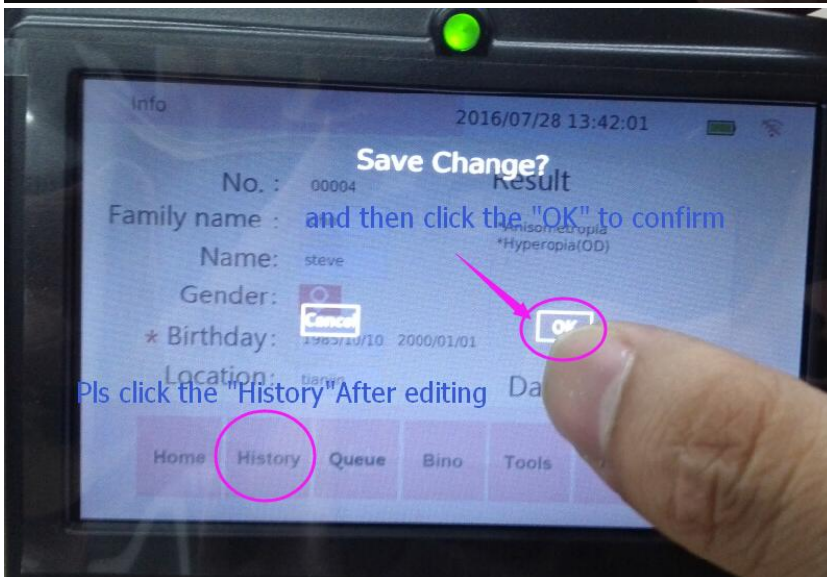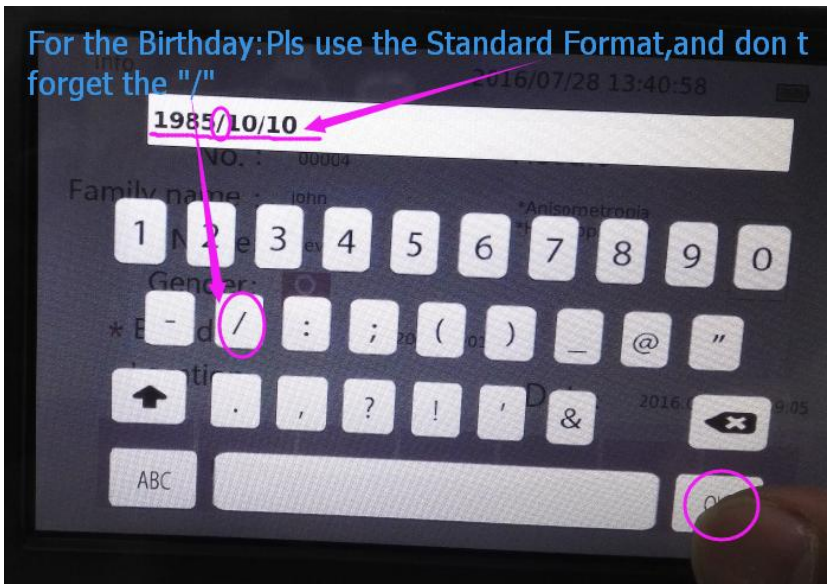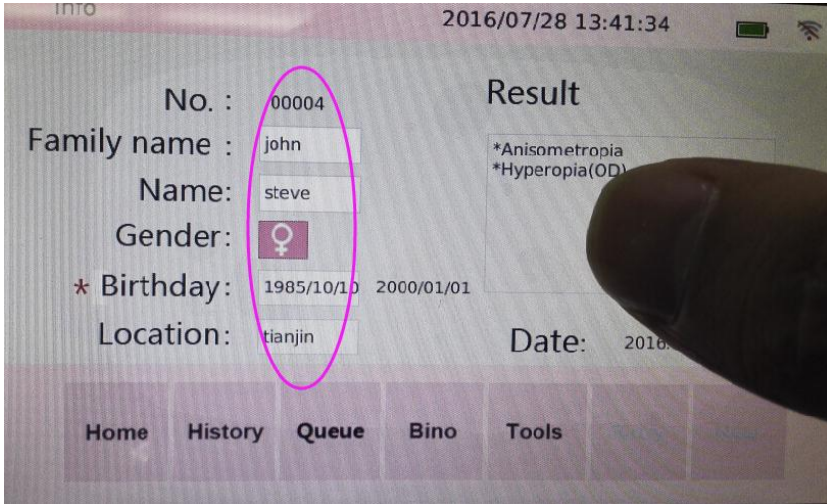


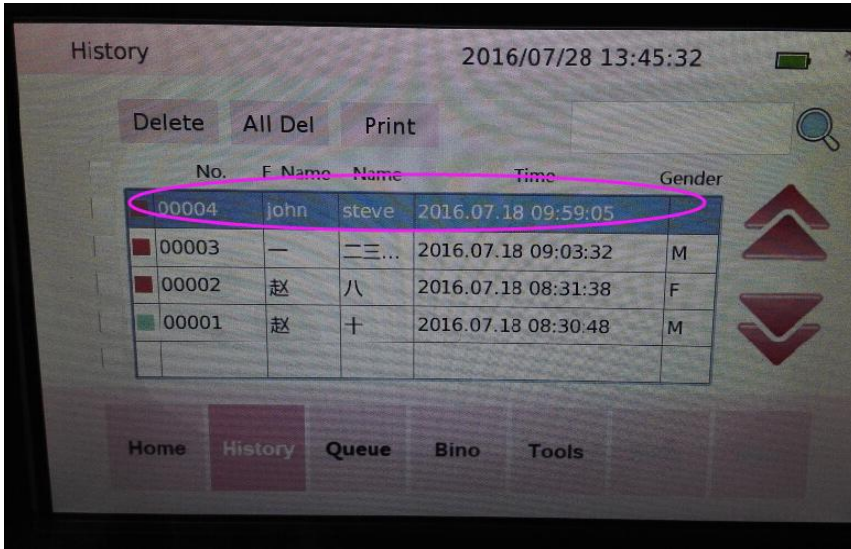

To Delete the Patient Case:


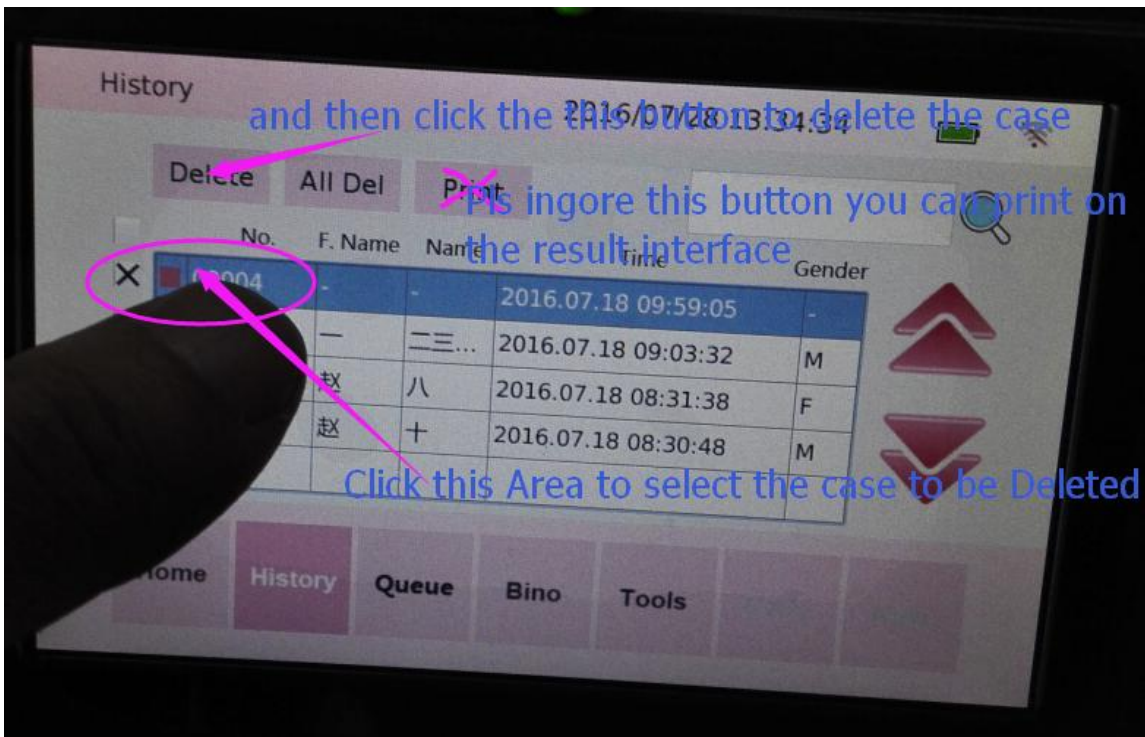


To Enter the “Result interface”:

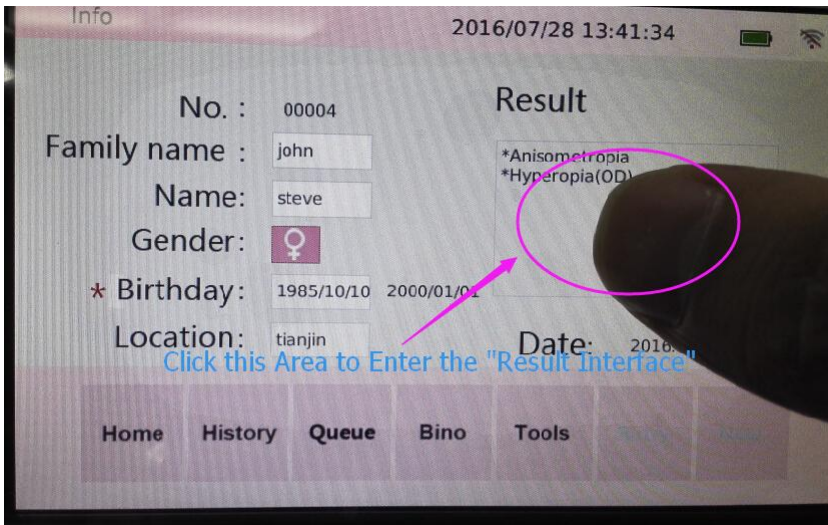


For Check the “Dioptroscoy,Red Reflex,and Examination of nystagmus”

We can check the “EYE”LOGO to check Exact Dioptroscoy,Red Reflex,and Examination of nystagmus.

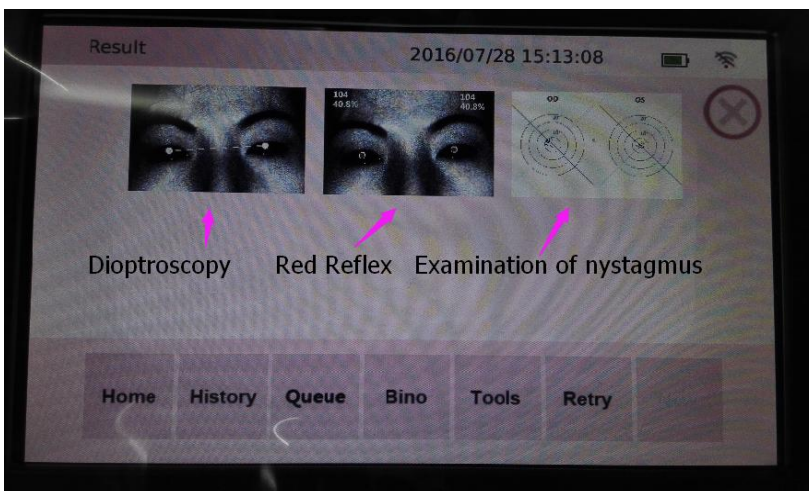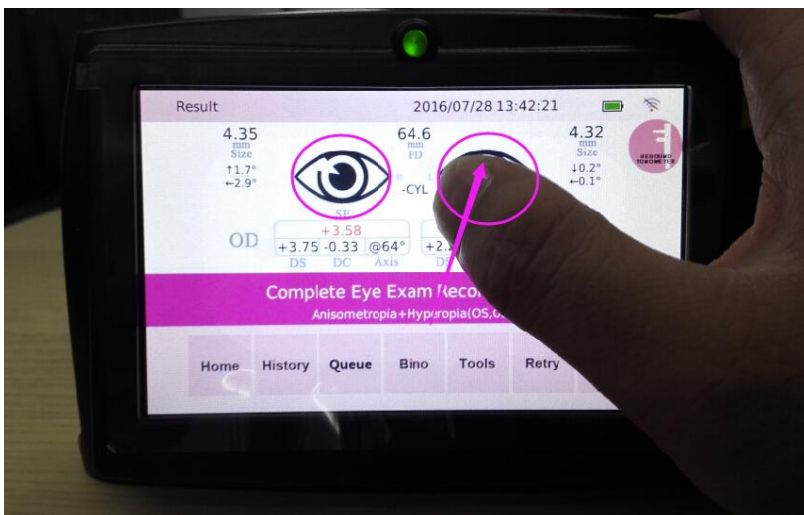

After We got the result,We can click the retry to do the next measurement.
PS:It is just available for the Same Screening Age range.

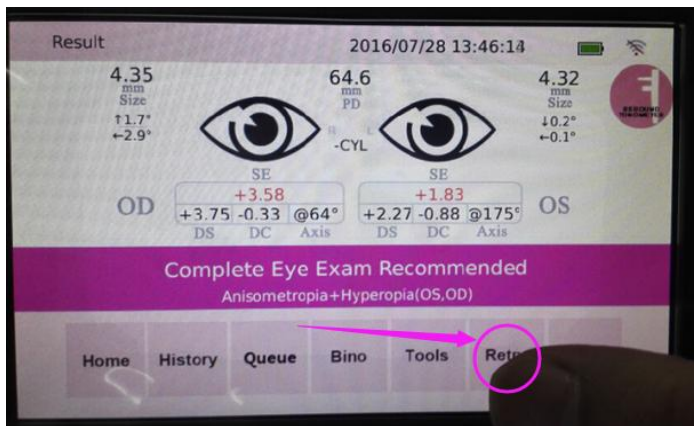


For Export the Patient Case by USB Disc:

Step 1: Insert the USB Disc on the port of Vision Screener,and choose the Choose “Tools”




Step 2:And then Click the LOGO of Data transfer

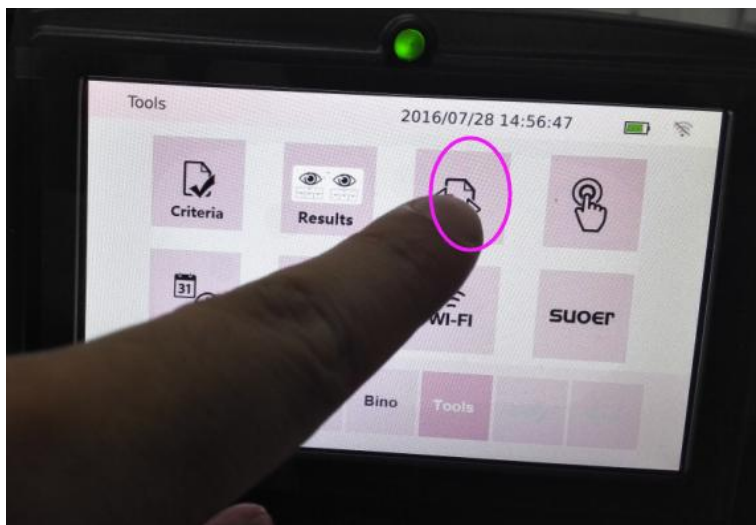


Step 3: wait for 10 Seconds ,and then choose the “Export”



Step 4: Choose the Format:

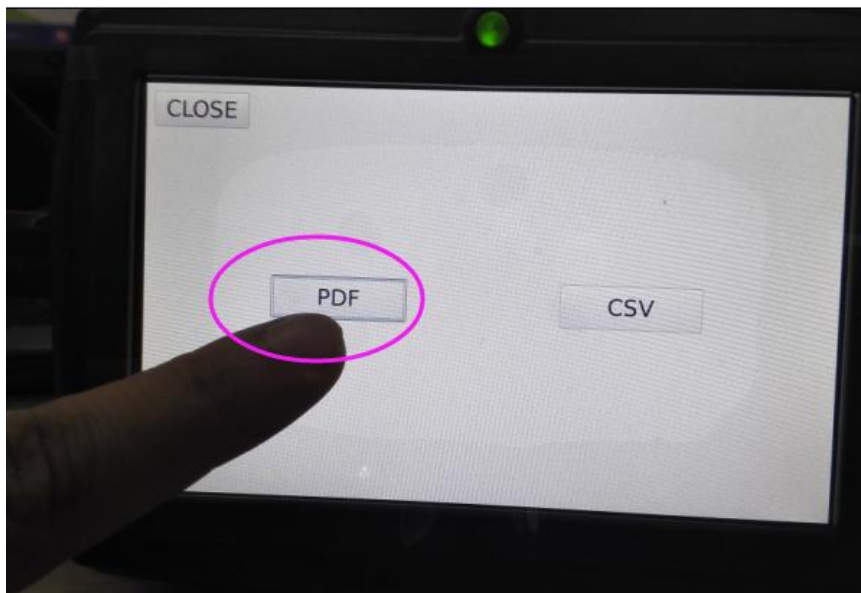


Step 5: Click “Cancel”

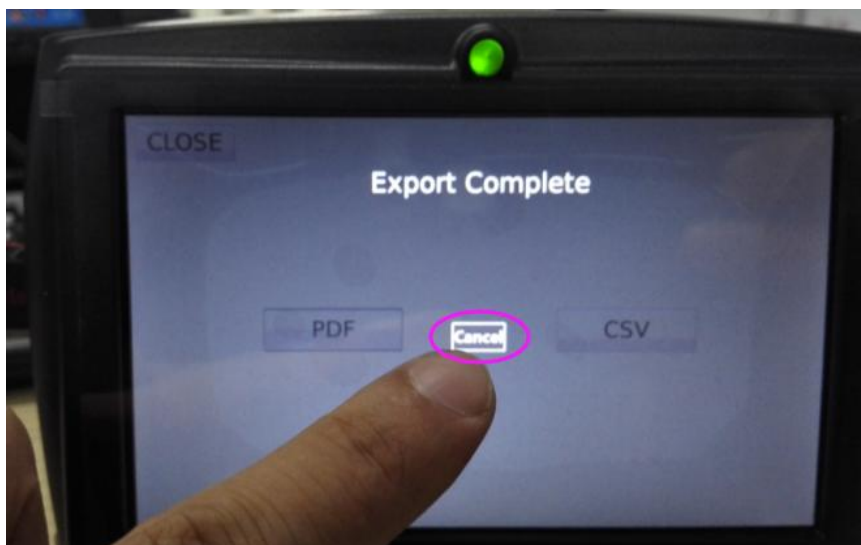

Step 6: Pull out the USB Disc and Insert it into the PC:

Step 7: Check the USB Disc on the PC and find the patient case and open it.

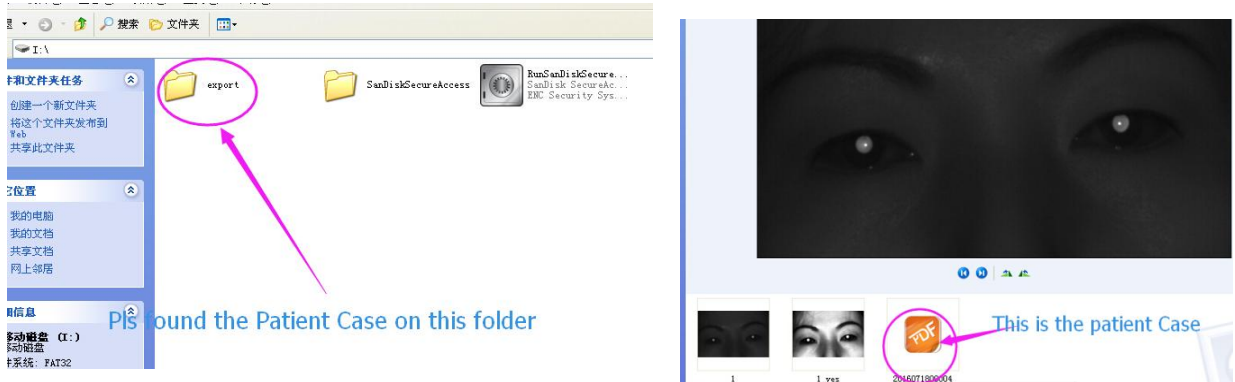

VISION SCREENER

ID: 2016071800004
TIME: 2016.07.18 09:59:05

Family Name: john	4.4	65	4.3
First Name: steve	<small>mm</small> Size	<small>mm</small> PD	<small>mm</small> Size
Sex: -	12°	-CYL	10°
BirthDay: 1985/10/10	--3°		-0°
Position: tianjin			

<R>

SE		
+3.75	-0.25	64°
S	C	A

<L>

SE		
+2.25	-1.00	175°
S	C	A

Dioptrics:

	<R>	<L>
Myopia:	✓	✓
Hyperopia:	✗	✓
Astigmatism:	✓	✓
Gaze:	✓	✓
Anisocoria:		✓
Anisometropia:		✗
Asymmetric:		✓


Red Reflex:

	<R>	<L>
Reflex Light:	40.8%	40.8%
Difference:		0.0%

Examination of nystagmus:

	<R>	<L>
Horizontal Range:	4.1°	4.2°
Horizontal Speed:	128.1°/s	153.1°/s
Vertical Range:	5.2°	4.9°
Vertical Speed:	162.5°/s	131.2°/s

SHTOPVIEW

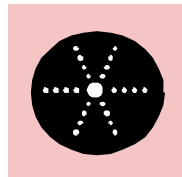

Ocular Screening SW-800





Ocular Screening

Optometry/Screening Easy as snapshot

SW-800



Ocular Screening



- **Binocular refractive Detection**
- **Pupil diameter measurement**
- **Work distance 1m**
- **Automatic detection**
- **Screening takes 1 Seconds**
- **Handheld devices, light-weight**
- **Replaceable rechargeable battery**
- **Non-invasive measurement**
- **Output test report**



Ocular Screening

Optometry/Screening Easy as snapshot

SW-800



Ocular Screening

Optometry

- § Binocular Detection
- § DS
- § DC
- § Axis angle
- § Pupil size
- § Interpupillary distance
- § Direction of fixation
- § Record the abnormal reflection of the light

Screening

- § Myopia
- § Hyperopia
- § Astigmatism
- § Strabismus
- § Pupil size vary
- § Anisometropia
- § Asymmetric fixation
- § The abnormal reflection of the light






Ocular Screening

Optometry/Screening Easy as snapshot


SW-800

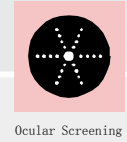


Ocular Screening

The whole optometry process 3'-5'


Focus

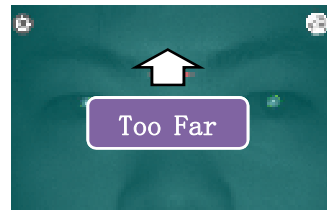


Measurement

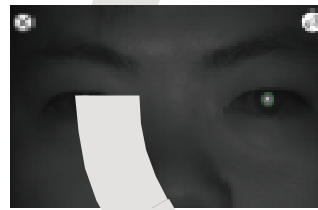

Calculation

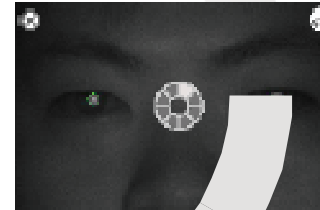

Result

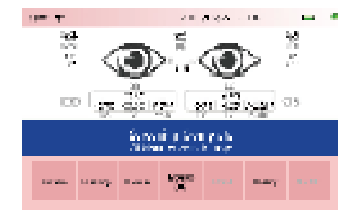

Ultrasonic Ranging & Auto Focus
Screening is performed
from 1 m away





Ocular Screening

Optometry/Screening Easy as snapshot

SW-800



Ocular Screening

标准 Standard 2015/06/25 14:38:43


年龄(月) AGE(Month)	Anisometropia		Myopia		Hyperopia		瞳孔大小差 Pupil size difference	固视 Fixation 垂直方向 (Vertical Direction)	固视 Fixation 鼻侧方向 (Nasal side Direction)	固视 Fixation 颞侧方向 (Temporal Direction)	固视 Fixation 不对称 Fixation Asymmetry
	屈光参差	散光 AST	近视	远视	瞳孔大小差	固视 垂直方向	固视 鼻侧方向	固视 颞侧方向	固视 不对称		
6-12	1.5	2.25	2	3.5	1	8	5	8	8		
12-36	1	2	2	3	1	8	5	8	8		
36-72	1	1.75	1.25	2.5	1	8	5	8	8		
72-240	1	1.5	1	2.5	1	8	5	8	8		
240-1200	1	1.5	0.75	1.5	1	8	5	8	8		



Ocular Screening


Optometry/Screening Easy as snapshot

SW-800

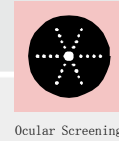

Ocular Screening

The abnormal reflection of the eye effecting by the infrared light can be screened out

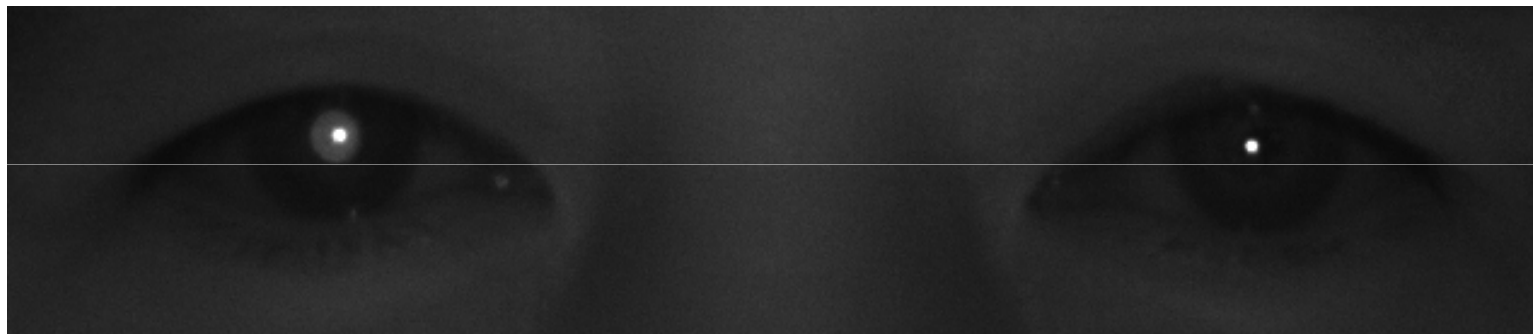

Normal


Abnormal

Abnormal situation: The abnormal reflection of the eye effecting by the infrared light needs to be checked further, Suspected Cataract, Vitreous opacity, and Retinal tumor, etc.



Ocular Screening SW-800

Optometry/Screening Easy as snapshot

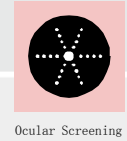


Ocular Screening





Ocular Screening SW-800

Optometry/Screening Easy as snapshot

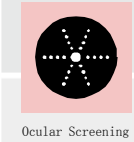

Complete Eye Exam Recommended

Myopia (OD)

Vision Screening Summary ID:

姓名: XXX

性别: 女

出生日期: 2001.8.25

测试地点:

vision screen should be conducted regularly as eyes may change over time

5.4 mm

$\frac{5.4}{32}$

OD

5.5 mm

$\frac{5.5}{32}$

OS

-1.50		
OD	OS	Axis
-1.50	+1.50	@148

-1.00		
OD	OS	Axis
-1.26	+0.50	@43

Complete Eye Exam Recommended

Myopia (OD)

	OD	OS		
近视:	✗	✗	瞳孔大小不等:	✓ ✓
远视:	✓	✓	屈光参差:	✓ ✓
散光:	✓	✓	圆锥不对称:	✓ ✓
斜视:	✓	✓		

注: 视力检查结果不能代替专业眼科医生的全面检查!

2015.07.03



**WIFI transmission
print report**



Ocular Screening

Optometry/Screening Easy as snapshot

SW-800



Ocular Screening

Technical data

Operation mode: Bino/Mono

Optometry: Automatic

DS: Range: -7.50D~+7.50D

Resolution: 0.25D/0.01D

Accuracy: $\pm 0.50D$

DC: Range: 0.00D~3.00D

Resolution: 0.25D/0.01D

Accuracy: $\pm 0.50D$

Axis: Range: $1^\circ \sim 180^\circ$

Resolution: 1°

Accuracy: $\pm 5^\circ$

Pupil size: Range: 4.0 mm~9.0 mm

Resolution: 0.1mm Accuracy: ± 0.1 mm

Pupil distance: Range: 35 mm~80 mm

Resolution: 1mm Accuracy: ± 1 mm

Gaze: $0^\circ \sim 20^\circ$

Measuring distance: $1\text{ m} \pm 5\text{ cm}$

Time per measurement: ~1s

Fixation target: Light flash, attractive sound

Data interface: Wi-Fi, USB

Printer interface: USB

Battery: Rechargeable lithium batteries, 6 hours of duration, Replaceable

Size: 180mm \times 130 mm \times 110mm

Display: 5 inch touch screen

Weight: 0.8Kg

Optional accessories: Camera tripods, printer, etc





Fiscal Year 2021

FDA Registration Confirmation

Manufacturer: SHANGHAI TOP VIEW INDUSTRIAL CO.,LTD

Address: No.3388 GongHeXin Road,JingAn Shanghai , CN 200436

US Agent: Regrek LLC

Address: 19 Holly Cove Ln.,Dover, Delaware, 19901, UNITED STATES

The facility registration and device listing information:

Registration Number:3012707847 **Owner/Operator Number:** 10059986

Device Listing#: See annex

This attestation does not denote endorsement or approval of the attestation-holder's device or establishment by the U.S. Food and Drug Administration.

Pursuant to 21 CFR 807.39, "Registration of a device establishment or assignment of a registration number does not in any way denote approval of the establishment or its products. Any representation that creates an impression of official approval because of registration or possession of a registration number is misleading and constitutes misbranding." The U.S. Food and Drug Administration does not issue an attestation of registration, nor does the U.S. Food and Drug Administration recognize an attestation of registration.


Authorized Signature(s)

Signature:

Place: China/ Shanghai

Expiration Date: Dec. 31. 2021



FDA Registration Confirmation

ANNEX:

Listing No	Code	Device Name
D345115	HKO	Refractometer, ophthalmic (TPV800 handheld refractor; TPV series auto refractometer)
D345116	HKN	Refractor, manual, non-powered, including phoropter (phoropter)
D345117	HKZ	Sterilizer, tonometer (TPV500 tonometer)
D345118	PJZ	Camera, ophthalmic, general-use (Hand-held Fundus Camera; TPV series Digital Fundus Camera)
D345119	PUE	Biomicroscope, slit-lamp, AC-powered, exempt (TPV series slit lamp)
D345120	HRJ	Table, instrument, powered, ophthalmic (Ophthalmic unit)
D345121	HLM	Instrument, measuring, lens, ac-powered (TPV series lens meter)
D345122	HPA	Frame, trial, ophthalmic (trial frame)
D345123	HPC	Set, lens, trial, ophthalmic (trial lens set)
D345124	HOX	Chart, visual acuity (visual charter)
D345125	HPT	Perimeter, automatic, ac-powered (perimeter)
D345341	HOS	Projector, ophthalmic (projector charter)
D345342	HLH	Pupillometer, manual (pupillometer)
D345343	HRM	MICROSCOPE, OPERATING & ACCESSORIES, AC-POWERED, OPHTHALMIC (operating microscope)

D345344	KZA	DEVICE, VEIN LOCATION, LIQUID CRYSTAL (vein finder)
D366893	HLJ	Ophthalmoscope, battery-powered (ophthalmoscope retinoscope)
D366894	MDM	INSTRUMENT, MANUAL, SURGICAL, GENERAL USE (ophthalmic surgical instrument)
D366895	MMQ	Topographer, corneal, ac-powered (cornealtopographer)

END OF THE ANNEX

 CEO
Authorized Signature(s)

Signature:

Place: China/ Shanghai

Expiration Date: Dec. 31. 2021



Alibaba.com verified Supplier



Assessment Report

Presented to

Shanghai Top View Industrial Co., Ltd.

上海拓扑威实业有限公司

Gold Supplier & Assessed Company Relationship:	<input checked="" type="checkbox"/> Self-owned <input type="checkbox"/> Wholly Owned <input type="checkbox"/> Shareholder/Partner <input type="checkbox"/> Cooperation Partner
Company Address	Room 906, Yongding Plaza, No. 3388, Gonghe New Road, Jingan District, Shanghai, China
City / Country:	Shanghai / China
Consigner of Assessment:	Alibaba
Gold Supplier Member ID:	shtopview
Gold Supplier Company Name:	Shanghai Top View Industrial Co., Ltd.
Contact Person:	Mr. Tao Shangguan
Phone Number:	0086-21-61178155
Fax Number:	N/A
Email:	info@shtopview.com
Website Address (URL):	https://shtopview.en.alibaba.com

Service Provided by Intertek

Report No.: 22069995_T





Alibaba.com Verified Supplier
Assessment Report

Report No.: 22069995_T

Report Number:	22069995_T	Assessment Type	Trading Assessment
Date of Assessment:	31/Aug./2021	Report Date:	31/Aug./2021
Assessor's Name:	Zad Zhang	Validity Period:	01/Sep./2021 - 31/Aug./2022
Reviewed By:	Mack Long	Online Verification:	https://www.intertek.com.cn/ASVService/Home/IndexCN

Important Notes:

Intertek's Disclaimer:

This report reflects our findings for the particularly concerned company on the date of our service only. This report does not discharge or release the factory/sellers/suppliers from their commercial, legal or contractual obligations with buyers in respect of products provided by the factory/sellers/suppliers. Any reader other than the party for whom this report has been specifically issued is hereby informed that the general conditions of service of contain liability limitation provisions.

Alibaba.com's Disclaimer:

This report has been independently prepared by a third party verification agency. The report is provided on an "AS IS" and "AS AVAILABLE" basis. The information contained in the report may change without notice and no obligation is assumed to update the information after publication. Alibaba (including its affiliates) is not involved in the preparation of the report. Publishing of the report by no means that Alibaba endorses or recommends the report. Therefore, Alibaba cannot be held responsible for any inaccuracy or omission in the report.

To the maximum extent permitted by law, Alibaba expressly disclaims any warranty, expressed or implied, in regard to the report, including any implied warranty of merchantability, fitness for a particular purpose or non-infringement. Your use of or reliance on this report will be at your own discretion and risk. To the maximum extent permitted by law, in no event shall Alibaba be liable for any party for any direct, indirect, punitive, incidental, special or consequential damage costs, expenses, legal fees or losses or any damages whatsoever arising out of or in any way connected with any use of the report even if advised of the possibility of such damages.

Report No:	22069995_T	Report date:	31/Aug./2021	Assessed By	Zad Zhang	
CONFIDENTIAL		All Rights Reserved			Page No:	2 of 25



Contents

Section 1: Company Overview	5
1.1 Legal Validity.....	5
Section 2: Human Resources	6
2.1 Company Chart.....	6
2.2 Employee Headcount	6
2.3 Management.....	6
Section 3: Current Export Situation	7
Section 4: Export Business Capacity	9
4.1 Market Distribution (Previous 12 Months)	9
4.2 Main Clients.....	10
Section 5: Quality Assurance	11
5.1.1 Quality Management System Certification	11
5.1.2 Product Certification	11
5.2 Supplier Management	11
5.3 After Sales Service	12
5.4 Overseas After Sales Service.....	12
Section 6: R & D Capacity	13
6.1 Current Situation	13
6.2 R&D Real Case Description.....	14
6.3 Design Process.....	15
6.4 Design Devices	15
Section 7: Company Development / Expansion Plans	16
Section 8: Certification & Photos	17
Section 9: Company and Product Samples	19
Section 10: Competitive Advantages	21
10.1 Product Group Capacity	21

Report No:	22069995_T	Report date:	31/Aug./2021	Assessed By	Zad Zhang	
CONFIDENTIAL		All Rights Reserved			Page No:	3 of 25



10.2 Real Case for Lower MOQ & Lead Time21

10.3 Real Case for Large Contract22

10.4 Overseas after-sales service capacity.....22

10.5 After-sales service capacity22

Section 11: Service Capabilities24

11.1 Experience with Large-scale Procurement Contracts24

11.2 Overseas Showroom24

11.3 Offline Trade Show24

Section 12: Supply Chain Capability25

12.1 Centralized Procurement25

12.2 Overseas Warehouse25



Section 1: Company Overview

Company Overview			
1.1 Legal Validity			
Does the company have a valid business license?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	Business License Number:	9131011506602810 2F
Year Established:	18/Apr./2013	Validity Period:	18/Apr./2013- 17/Apr./2033
Export Experience:	8 Years	Industry Experience:	8 Years
Registered Address:	Room 150, Building 3, No. 2558, Zhouzhu Road, Pudong New Area, Shanghai, China		
Company Address:	Room 906, Yongding Plaza, No. 3388, Gonghe New Road, Jingan District, Shanghai, China		
Is It Listed Company?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	Company Stock Code:	N/A
Annual review conducted by the Industrial & Commercial Bureau?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	Reviewed By:	Market Supervision Administration Bureau of Pudong New Area, Shanghai
Registered Capital:	RMB 2,000,000		
Corporate Representative:	Mr. Tao Shangguan		
Industry:	Optical Instrument		
Business Type:	<input type="checkbox"/> Manufacturer <input checked="" type="checkbox"/> Trading Company <input type="checkbox"/> Manufacturer & Trading Company		
Type of Ownership:	<input checked="" type="checkbox"/> Private Owner <input type="checkbox"/> Public Company <input type="checkbox"/> Joint Venture <input type="checkbox"/> Stated Owned <input type="checkbox"/> Sole Proprietorship <input type="checkbox"/> Other		
Products /Service:	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer		
1.2 Company Building Information			
Certification Type:	<input type="checkbox"/> Land Certification <input type="checkbox"/> Real Estate Certification <input checked="" type="checkbox"/> Lease Agreement <input type="checkbox"/> Factory Officer Claimed		
Total Building Size:	<u>326</u> m2		
Office Size:	<u>326</u> m2		



Section 2: Human Resources

Human Resources				
2.1 Company Chart				
<pre>graph TD; GM[GM] --- Marketing[Marketing Dept.]; GM --- Sales[Sales Dept.]; GM --- Finance[Finance Dept.]; GM --- Admin[Admin Dept.];</pre>				
2.2 Employee Headcount				
Department Name	Full Time Employee(s)	Part-Time Employee(s)	Total	
GM	1	0	1	
Marketing Dept.	4	0	4	
Sales Dept.	8	0	8	
Finance Dept.	1	0	1	
Admin Dept.	1	0	1	
Total Number:	15	0	15	
2.3 Management				
Department	Name	Title	Education	Year(s) in Current Company
Confidential	Confidential	Confidential	Confidential	Confidential



Section 3: Current Export Situation

Current Export Situation							
There are <u>8</u> foreign trading employee(s) in the company.							
Working Experience	Headcount	Accepted Language	Listening & Speaking	Reading & Writing			
Over 30 Years	0	English	Yes	Yes			
21-30Years	0	N/A	N/A	N/A			
11-20 Years	0	N/A	N/A	N/A			
6-10 Years	0	N/A	N/A	N/A			
2-5 Years	8	N/A	N/A	N/A			
Less than 2 years	0	N/A	N/A	N/A			
Does the company have a valid export license?			<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No				
Export License Registration No.:			01314518				
Total Revenue (Previous Year, USD):			Confidential				
Total Export Revenue (Previous Year, USD):			c				
Estimated Export Revenue (Current Year, USD):			Confidential				
Trade Agents Employed Overseas:			<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No				
Nearest Port:			Shanghai Port				
Accepted Payment Terms			<input checked="" type="checkbox"/> FOB <input checked="" type="checkbox"/> CIF <input checked="" type="checkbox"/> EXW <input checked="" type="checkbox"/> CFR				
Accepted Payment Type:			<input checked="" type="checkbox"/> L/C <input checked="" type="checkbox"/> T/T <input checked="" type="checkbox"/> Credit Card <input checked="" type="checkbox"/> Cash <input checked="" type="checkbox"/> West Union <input checked="" type="checkbox"/> Money Gram <input checked="" type="checkbox"/> Paypal <input checked="" type="checkbox"/> Moneybookers				
Average lead time from product order confirmation to production delivery (products exiting the factory):							
Product Category	Num	Unit					
Auto Lens Meter	3	Days					
Slit Lamp	3	Days					
3D Auto Lens Edger	3	Days					
Rebound Tonometer	3	Days					
Auto Refractometer	3	Days					
Average Sampling Time							
Product Category	Lead Time						
Auto Lens Meter	3 Days						
Slit Lamp	3 Days						
3D Auto Lens Edger	3 Days						



Alibaba.com Verified Supplier
Assessment Report

Report No.: 22069995_T

Rebound Tonometer	3 Days
Auto Refractometer	3 Days
The Shortest Sampling Time	
Product Category	Shortest Lead Time
Auto Lens Meter	3 Days
Slit Lamp	3 Days
3D Auto Lens Edger	3 Days
Rebound Tonometer	3 Days
Auto Refractometer	3 Days



Section 4: Export Business Capacity

Export Business Capacity			
4.1 Market Distribution (Previous 12 Months)			
Market	Main Product(s)	Revenue (USD)	Total Revenue (%)
North America	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5
South America	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	50
Eastern Europe	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	2
Southeast Asia	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5
Africa	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5
Oceania	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	2
Mid East	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5
Eastern Asia	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5
Western Europe	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	10
Central America	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	2



Northern Europe	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	2
Southern Europe	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	2
South Asia	Auto Lens Meter, Slit Lamp, 3D Auto Lens Edger, Rebound Tonometer, Auto Refractometer	Confidential	5

4.2 Main Clients

Client Name	Main Product(s)	Total Revenue (%)
Confidential	Confidential	Confidential



Section 5: Quality Assurance

Quality Assurance						
5.1.1 Quality Management System Certification						
Certification	Certified By	Certificate No.	Business Scope	Validity Date		
N/A	N/A	N/A	N/A	N/A		
5.1.2 Product Certification						
Certification	Certified By	Certificate No.	Product Name & Model No.	Validity Date		
N/A	N/A	N/A	N/A	N/A		
5.2 Supplier Management						
Item	Content	Observations /Comments				
5.2.1	Does the company have a supplier assessment procedure?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No				
5.2.2	Does the company have an updated list of approved suppliers?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No				
5.2.3	Has the company established and implemented a standard procedure for purchasing contract review and approval?	<input checked="" type="checkbox"/> Yes, with written standard procedures <input type="checkbox"/> Yes, with written procedure but lack of consistent standard <input type="checkbox"/> No				
5.2.4	Does the company keep its supplier assessment reports?	<input checked="" type="checkbox"/> Yes, assessment reports are available for more than 3 years <input type="checkbox"/> Yes, assessment reports are available for the last 1-3 years <input type="checkbox"/> Yes, assessment reports are available for the previous 12 months <input type="checkbox"/> No				
5.2.5	Are the company's purchasing documents sufficient to ensure product safety control and their customers' requirements?	<input checked="" type="checkbox"/> Yes, the purchasing document includes all the information required <input type="checkbox"/> Yes, however the purchasing document includes incomplete information <input type="checkbox"/> No				
5.2.6	Is there a procedure to conduct random product inspections after final packaging?	<input checked="" type="checkbox"/> Yes, with clear standard and written inspection records <input type="checkbox"/> Yes, with inspection records but no procedures <input type="checkbox"/> Yes, with procedures but no inspection records <input type="checkbox"/> No, inspections are not necessary				
Report No:	22069995_T	Report date:	31/Aug./2021	Assessed By	Zad Zhang	
CONFIDENTIAL		All Rights Reserved			Page No:	11 of 25



5.3 After Sales Service		
Item	Content	Observations /Comments
5.3.1	Is customer feedback, including complaints, clearly recorded and maintained?	<input checked="" type="checkbox"/> Yes, with a standard feedback form and records <input type="checkbox"/> Yes, with a standard feedback form but no records <input type="checkbox"/> Yes, with records but no standard feedback form <input type="checkbox"/> No
5.3.2	Are there any clear procedures for handling customer complaints?	<input checked="" type="checkbox"/> Yes, with clear procedures and written records <input type="checkbox"/> Yes, with clear procedures but no written records <input type="checkbox"/> Yes, with written records but no clear procedures <input type="checkbox"/> No
5.3.3	Is there a closed-loop corrective action system in place?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No
5.3.4	Can finished/package products be traced by lot identification to the appropriate raw material test reports?	<input checked="" type="checkbox"/> Yes, with procedures to trace raw materials <input type="checkbox"/> Yes, main raw material can be traced <input type="checkbox"/> No, only the production date can be traced <input type="checkbox"/> No
5.3.5	Is there a product alert and recall procedure?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No
5.3.6	Do you have a complete after sales service capability?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No
5.4 Overseas After Sales Service		
5.4.1	Is it possible to provide expatriate engineer services?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No
5.4.2	If possible provide expatriate engineer, what kind of onsite after-sales services are included?	<input checked="" type="checkbox"/> Debugging <input checked="" type="checkbox"/> Maintain <input checked="" type="checkbox"/> Repair <input type="checkbox"/> Other



Section 6: R & D Capacity

R&D Capacity				
6.1 Current Situation				
There is <u>0</u> R&D engineer(s) in the company.				
Education Level	Headcount		Work Experience	Headcount
Doctorate	0		Over 30 Years	0
Post-Graduate	0		21-30 Years	0
Graduate	0		11-20 Years	0
Junior College	0		6-10 Years	0
Technical School	0		2-5 Years	0
High School	0		Less than 2 years	0
Patent Situation				
Patent No.	The Name of the Patent		The Patent Type	Available Date
N/A	N/A		N/A	N/A
Brand Situation				
Registration/ap plication No.	Brand Name	For Approval to Use Goods	Validity Date	Ref.
27928760	Refer to Photo	Category: 9	14/Nov./2018-13/Nov./2028	Photo in Section 8 (Trademark Photos)
The Average Time For New Products Launched				
Product Category		Num	Unit	
N/A		N/A	N/A	
The Shortest Time For A New Item Launched				
Product Category		Num	Unit	
N/A		N/A	N/A	
Does the company provide ODM service for others?		<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		
Are there relevant design input/output, review, and verification documents available for the assessment company?		<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		
Based on inspection, are R & D employees equipped with adequate specialized equipment?		<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		
If yes, please list all key equipment used:		N/A		



Do R& D employees use any specific software for designing new products?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
If yes, please list the main software used:	N/A
Please list all certifications and/or qualifications of the R & D department:	N/A
Has the company established standard design procedures for new products?	<input type="checkbox"/> Yes, with clear written instructions <input type="checkbox"/> Yes, without written instructions <input checked="" type="checkbox"/> No
Have the designed products been internal verified or validated?	<input type="checkbox"/> Yes, with clear written records <input type="checkbox"/> Yes, only part written records <input type="checkbox"/> Yes, without written records <input checked="" type="checkbox"/> No
Have the designed products been tested by a third-party inspection body?	<input type="checkbox"/> Yes, all designed products have been tested <input type="checkbox"/> Yes, only part of designed products have been tested <input checked="" type="checkbox"/> No
Are the designed products confirmed by the customers?	<input type="checkbox"/> Yes, all designed products have been confirmed <input type="checkbox"/> Yes, part of designed products have been confirmed according to client's requirements <input checked="" type="checkbox"/> No
Does the company has qualification requirements for designers?	<input type="checkbox"/> Yes, with written job description <input type="checkbox"/> Yes, without written job description <input type="checkbox"/> No, but at least two years design experience is needed <input checked="" type="checkbox"/> No
Are the designers' qualifications recognized by the company?	<input type="checkbox"/> Yes, with written records <input type="checkbox"/> Yes, without written records <input checked="" type="checkbox"/> No
What level of design services are provided?	<input checked="" type="checkbox"/> Only add logo/change color/material <input type="checkbox"/> Sample processing <input type="checkbox"/> Graphic processing <input type="checkbox"/> Create an entirely new product
New Products Launched quantity for each year	
Product Category	Num Unit
N/A	N/A N/A
6.2 R&D Real Case Description	
Customer's Name	N/A



Customer's Location	N/A	
Customer's Industry:	N/A	
Order's Requirement Description:	N/A	
6.3 Design Process		
Process 1	Process 2	Process 3
N/A	N/A	N/A
Description: N/A	Description: N/A	Description: N/A
6.4 Design Devices		
Device 1	Device 2	Device 3
N/A	N/A	N/A



Section 7: Company Development / Expansion Plans

Company Development / Expansion Plans		
Item	Company Development Action	Timeframe
1	The organization is going to develop more markets.	1 Year



Section 8: Certification & Photos

Certification & Photos

Certification & Photos -- Business License (Original)




Certification & Photos -- Business License (Duplicate) with Certificate / Records of Annual Inspection



Certification & Photos -- Permit for Opening Bank Account





Certification & Photos -- Import and Export Enterprise Registration

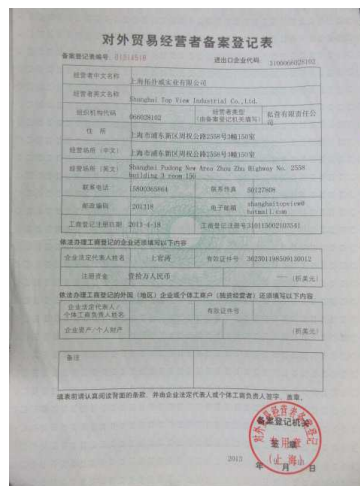

Certification & Photos -- Custom Clearance Registration Form

Certification & Photos -- N/A

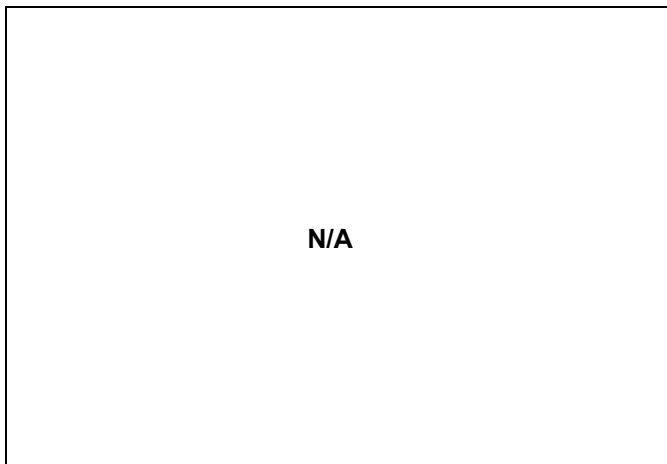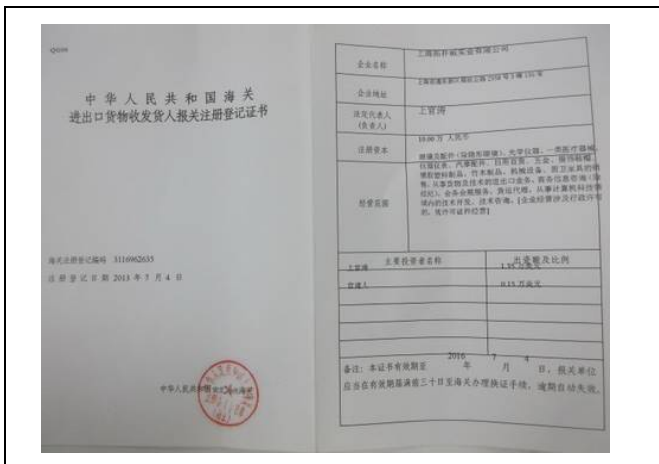

N/A

Patent Photos

Certification & Photos -N/A

Certification & Photos -N/A

N/A

N/A

Product Certification Photos

Certification & Photos --

Certification & Photos --

N/A

N/A

Quality Management System Certification Photos

Certification & Photos --

Certification & Photos --

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
N/A

N/A

Trademark Photos





Certification & Photos -- Trademark

Certification & Photos -- N/A



N/A

Section 9: Company and Product Samples

Company and Product Samples	
Company Gate	Office
	
Product Sample- Auto Lens Meter	Product Sample- Slit Lamp
	
Product Sample-3D Auto Lens Edger	Product Sample- Rebound Tonometer



Product Sample- Auto Refractometer



N/A



N/A



Section 10: Competitive Advantages

10.1 Product Group Capacity		
10.1.1 Products Sold (Within 12 Months)		
Products Name	Quantity	Revenue (USD)
Auto Lens Meter	400 Sets	Confidential
Slit Lamp	150 Sets	Confidential
3D Auto Lens Edger	70 Sets	Confidential
Rebound Tonometer	300 Sets	Confidential
Auto Refractometer	500 Sets	Confidential
10.1.2 Suppliers Cooperated With (Within 12 Months)		
No. of cooperation suppliers (total)	Confidential	
No. of suppliers (which cooperated over 2 times)	Confidential	
No. of provinces which cooperation suppliers belong to	Confidential	
Would the company like to provide design solution service for integration project?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	
If yes, these projects include	According to client requirement	
Would the company like to provide a total solution for purchasing?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> Part, _____ <input type="checkbox"/> No	
If yes, please describe it	According to client requirement	
10.1.3 Real Case Description:		
Customer Name	Confidential	
Customer Country	Confidential	
Customer Region	Confidential	
Products Category	Confidential	
Order Value (USD)	Confidential	
Order Processing Process	Confidential	
Customer's Feedback	Confidential	
10.2 Real Case for Lower MOQ & Lead Time		
Products Name	MOQ (In the last 12 Months)	Shortest Lead Time
Auto Lens Meter	Confidential	Confidential
Slit Lamp	Confidential	Confidential
3D Auto Lens Edger	Confidential	Confidential
Rebound Tonometer	Confidential	Confidential



Auto Refractometer	Confidential	Confidential
MOQ (In the last 12months)	<input checked="" type="checkbox"/> Less than 10 <input type="checkbox"/> 10-20 <input type="checkbox"/> 20-50 <input type="checkbox"/> 50-100 <input type="checkbox"/> 100-300 <input type="checkbox"/> More than 300	

10.3 Real Case for Large Contract

Products Name	Order (In the past 12 Months)	Shortest Lead Time
Auto Lens Meter	Confidential	Confidential
Slit Lamp	Confidential	Confidential
3D Auto Lens Edger	Confidential	Confidential
Rebound Tonometer	Confidential	Confidential
Auto Refractometer	Confidential	Confidential

10.4 Overseas after-sales service capacity

If yes, what onsite after-sales services are included	N/A
Equipment -installation, maintenance and other services	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
Technical advice	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
Personnel training	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
Other:	N/A
Average response time:	
Num	Unit
N/A	N/A

10.5 After-sales service capacity

Average Guarantee Time		
Product Category	Num	Unit
Auto Lens Meter	1	Year
Slit Lamp	1	Year
3D Auto Lens Edger	1	Year
Rebound Tonometer	1	Year
Auto Refractometer	1	Year
The Longest Guarantee Time		
Product Category	Num	Unit
Auto Lens Meter	1	Year
Slit Lamp	1	Year
3D Auto Lens Edger	1	Year



Alibaba.com Verified Supplier
Assessment Report

Report No.: 22069995_T

Rebound Tonometer	1	Year
Auto Refractometer	1	Year
Does the company accept small order?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	



Section 11: Service Capabilities

11.1 Experience with Large-scale Procurement Contracts		
Have you had a procurement contract with a Fortune 500 company?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	
Do you have an overseas onsite service center?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	
The country/region of your overseas service center	N/A	
11.2 Overseas Showroom		
Do you have an overseas showroom?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	
Country/Region	N/A	
11.3 Offline Trade Show		
Have you participated in offline trade shows?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	
Tradeshow name	CHINA (SHANGHAI) INTERNATIONAL OPTICS FAIR	Official images from the trade show
Date attended	11/Feb./2020	Confidential
Host Country/Region	Other China	



Section 12: Supply Chain Capability

12.1 Centralized Procurement	
In the past 12 months, have you sold products from at least three sub-categories?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
12.2 Overseas Warehouse	
Do you have an overseas warehouse?	<input type="checkbox"/> Yes <input checked="" type="checkbox"/> No
Country/Region	N/A

-- End of Report --