

Instruction for Use

【Product Name】 Saliva Collection Kit

【Model】 TY180, TY180F, TY180D, TY002A-10.0, TY002B-10.0, TY002AF-10.0, TY002AD-10.0, TY003A-5.0, TY003B-5.0, TY003AF-5.0, TY003AD-5.0, TY004A-6.0, TY004B-6.0, TY004AF-6.0, TY004AD-6.0

【Content of Product】 consists of collection funnel, acquisition Pipeline, saliva transport medium.

【Scope of application】 It is used to collect, store and transport human oral saliva samples, only for in vitro analysis and detection purposes, not for therapeutic purposes.

【Principle and Advantage】 To collect saliva samples with a collection tube and then mix the samples with saliva preservation solution uniformly so that to ensure the integrity of the DNA&RNA in the saliva samples, and long-term storage at room temperature.

【Instructions for use】

- 1.30 minutes before collecting the sample, please rinse your mouth with water, fast after rinsing, and do not chew gum or drink alcohol, etc.
- 2.Relax and gently massage the sides of your cheeks, and spit saliva into the funnel until the amount of saliva (without bubbles) reaches the height of the saliva filling line.
3. Hold the sampling tube with one hand to ensure that the sampling tube is in an upright state. After spitting out saliva, pour the transport medium and mix up and down. Make sure the lid is tightly closed.
4. Turn upside down to ensure that saliva and transport medium are mixed.
5. Keep the collection tube upright, unscrew the funnel, and cover and tighten the collection tube with the collection tube cap.
6. Turn the tightened collection tube upside down and mix 10-15 times, put it in the packaging bag, and discard the funnel.

【Cautions】

- 1.For in vitro diagnosis only.
- 2.Do not use if it is expired,turbid or leaking.
- 3.Do not use if any damage of the package.
- 4.Each piece is for 1 person and used once.
- 5.Do not eat, drink, smoke, etc., 30 minutes before collecting samples.





【Store Under】 Be stored at room temperature and dry storage for 24 months before sampling









【Validity Period】 The kit can be stored at room temperature for 24 months.

【Manufacture Date】 See package

【Batch code】 See package

【Explanation of the labels graphics, symbols, and abbreviations】

	Manufacturer		Consult instructions for use
	Do not re-use		Keep away from sunlight

	Keep dry		Do not use if package is damaged.
	Batch code		Use-by date
	Date of manufacture		Authorized representative in the European Community
	For in vitro diagnosis only		Caution

【CE LOGO】

**Miraclean Technology Co.,Ltd.**

Address: Room 301, Building A, No.18, Rongshuxia
Industrial Zone, Tongxin Community, Baolong Street,
Longgang District, Shenzhen, 518116, Guangdong, P.R.
China

Tell: +86-0755- 89616773

Fax: +86-0755-89616775

Web: www.mantacc.com

**Share Info Consultant Service LLC Repräsentanzbüro**

Address: Heerdter Lohweg 83, 40549 Düsseldorf

Tel: 0049 1767 0057022

Emai: info@share-info.cn

DIMDI NO.: DE/0000047946

【Reference document】

[1] Wang Hui, Ren Health, Editor-in-Chief Wang Minggu, *Clinical Microbiology Test* (2015).

Beijing: People's Medical Publishing House, PP.133-151

[2] National Influenza Surveillance Plan (2017)

【Specification Version】 VR1.0 May. 2022